
# Smart Ring R1

User manual – EN


## 1. Package contents and description

- iGET Smart Ring R1
- Charging box
- USB-C cable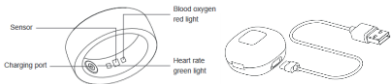

## 2. Switching on the device

Insert the ring into the charging box to switch it on. The box may be discharged, connect it to the charger before using it for the first time.

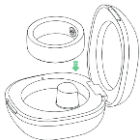### 3. Connecting to the app

Search and download the "iGET Smart Ring" app

Google Play: iGET Smart Ring

App Store: iGET Smart Ring

Or scan the code to download the app:

**Important: Please pair the ring in the app, not in Settings on your mobile phone.**

### 4. Charging

The red light will be on when charging and the green light will be on when fully charged.

 charging

 Fully Charge

## 5. Method of wearing

Keep the sensors on the inside of your finger for the most accurate measurement of heart rate and blood oxygen.

We recommend wearing the ring on your index finger, middle finger, or ring finger for best data tracking.

## 6. Description of functions

### Pedometer

The ring records your daily activity automatically. After syncing with the app, you can check your sports data on your smartphone.

**The measured data is indicative.**

### Heart rate, blood oxygen, stress level

After syncing with the app, you can check your heart rate data on your phone. **The measured data is only indicative.**

### Sports activities

You start the sports activity in the app - the ring records important data such as steps and heart rate. When the activity is over, you'll find the measurement result in the app.

**Note: If the sport time is less than 5 min, the data will not be saved. The ring does not have GPS, the measured data is only indicative.**

## Sleep

The ring records your daily sleep automatically. After syncing with the app, you can check your sleep data on your smartphone. **The measured data is indicative. They are measured between 18:00h in the evening and 12:00 in the morning.**

### 7. Safety notice:

- Small parts can cause choking.
- Do not expose the device to open flames.
- The device is not resistant to metal filings.
- Do not expose the device to magnetic radiation.
- Avoid extreme temperatures (-10°C, +40°C).
- Switch off the device in hospitals and other medical facilities if requested.
- Do not disassemble the device.
- Only use the device with certified equipment.
- Do not rely solely on your smart device during emergencies.
- In the event of any change in the dimensions of the device due to battery overheating, stop using the device until the battery is replaced with a new one.

Damage requiring professional repair:

If the following occurs, contact an authorized service center or supplier:

- Moisture has entered the equipment.
- The device is mechanically damaged.
- The equipment is overheating.

For households: The indicated symbol (crossed out bin) on the product or in the accompanying documentation means that used electrical or electronic products must not be disposed of together with municipal waste. In order to properly dispose of the product, hand it in at designated collection points, where they will be accepted free of charge. By disposing of this product correctly,

you help preserve valuable natural resources and help prevent potential negative impacts on the environment and human health that could result from improper waste disposal. Ask your local authority or nearest collection point for further details. Improper disposal of this type of waste may result in fines in accordance with regulations. Information for users on the disposal of electrical and electronic equipment (company and business use): For the correct disposal of electrical and electronic equipment, ask your dealer or supplier for detailed information. Information for users for the disposal of electrical and electronic equipment in other countries outside the European Union: The above symbol (crossed out bin) is only valid in the countries of the European Union. For the correct disposal of electrical and electronic equipment, request detailed information from your authorities or equipment dealer. Everything is expressed by the crossed-out container symbol on the product, packaging or printed materials. Apply for device warranty repairs at your dealer. In case of technical problems and questions, contact your dealer. Follow the rules for working with electrical devices. The user is not authorized to disassemble the device or replace any of its parts. There is a risk of electric shock when the covers are opened or removed. You also run the risk of electric shock if the device is assembled and reconnected incorrectly.

**Declaration of conformity:**

INTELEK LTD hereby declares that the Smart Ring R1 type of radio equipment complies with the essential requirements and other relevant provisions of Directive 2014/53/EU. The full text of the EU Declaration of Conformity is available on this website [www.iget.eu](http://www.iget.eu). The product is guaranteed for 24 months unless otherwise stated. The Declaration of Conformity can be downloaded from [www.iget.eu](http://www.iget.eu)

This device can be used in the following countries

AT	BE	CY	CZ	DK	EE	FI	MT	NL	PL	PT	SK
SI	ES	DE	GR	HU	IE	IT	LV	LT	GB	IS	LI
NO	CH	BG	HR	RU	RO	FR	SE	LU	TR		

Restrictions on use:

Italy - the conditions of use are described in the so-called Electronic Communications Code

Norway - the restriction in the 2.4 GHz band only applies to the area within 20 km of Ny-Ålesund. When using radio equipment in the frequency range 5150-5350 MHz, use is limited to indoor areas only.

Operation of radio equipment:

EN: This radio equipment may be operated in the Czech Republic under general authorisation VO-R/1/05.2017-2. SK: This radio equipment may be operated under a general authorisation, VPR - 02/2017.

RoHS: This device complies with the requirements on the restriction of the use of certain hazardous substances in electrical and electronic equipment (Government Regulation No. 481/2012 Coll., as amended by Government Regulation No. 391/2016 Coll.) and thus with the requirements of Directive (EC) 2011/65/EU of the European Parliament and of the Council on the restriction of the use of certain hazardous substances in electrical and electronic equipment. The RoHS declaration can be downloaded from [www.iget.eu](http://www.iget.eu).

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Copyright © 2024 INTELEK LTD, organizační složka All rights reserved.
Copyright © 2024 INTELEK LTD, organizační složka Všechna práva vyhrazena.
Copyright © 2024 INTELEK LTD, organizační složka Všetky práva vyhradené.
Copyright © 2024 INTELEK LTD, organizační složka. Wszelkie prawa zastrzeżone.
Copyright © 2024 INTELEK LTD, organizační složka Minden jog fenntartva.
Copyright © 2024 INTELEK LTD, organizační složka Alle Rechte vorbehalten.